

SET® and rainbow® Sensor & Cable Compatibility

Masimo Technology ▶		rainbow SET®		SET®		
Patient Cable ▶		RD rainbow SET®	rainbow® RC	RD SET®	M-LNC	Red LNC/LNC
rainbow® Sensor ▼	Connector ▶	RD15	M15	RD15	M15	DB9
RD rainbow SET-2 Disposables		Y	N	Y	N	N
RD rainbow Lite SET®-1 Disposables		Y	N	Y	N	N
rainbow® Disposables R		N	Y	N	Y	N
rainbow® Disposables R1		N	Y	N	Y	N
rainbow ReSposable® R2		N	Y	N	Y	N
rainbow® SpHb® Spot-check Reusables		N	Y	N	N	N
rainbow® SpCO® Reusables		N	Y	N	Y	N
SET® Sensor ▼						
RD SET Disposables & Reusables		Y	N	Y	N	N
M-LNCS Disposables & Reusables		N	Y	N	Y	N
LNCS Disposables & Reusables		N	N	N	N	Y

Minimum Software Compatibility Requirement[†]

Patient Cable ▶		RD rainbow SET®						RD SET®			
RD rainbow™/RD SET® Sensor ▼	Parameter ▶	SpHb®	SpMet®	SpCO®	ORi™	RRp®	PVi®	SET®	RRp**	PVi	SET®
RD rainbow SET-2 Disposables		7.4	7.1		7.C	7.9	4.8	No Min	7.9	4.8	No Min
RD rainbow Lite SET®-1 Disposables					7.C	7.9	4.8	No Min	7.9	4.8	No Min
RD SET Disposables & Reusables						7.9	4.8	No Min	7.9	4.8	No Min

Patient Cable ▶		rainbow® RC						M-LNC			
rainbow®/M-LNCS® Sensor ▼	Parameter ▶	SpHb	SpMet	SpCO	ORi	RRp	PVi	SET®	RRp	PVi	SET®
rainbow® Disposables R			7.1	7.1		7.9	4.8	No Min	7.9	4.8	No Min
rainbow® Disposables R1		7.4	7.1		7.C	7.9	4.8	No Min	7.9	4.8	No Min
rainbow ReSposable® R2		7.4	7.1		7.C	7.9	4.8	No Min	7.9	4.8	No Min
rainbow® SpHb® Spot-check Reusables*		7.4	7.4	7.4***		7.9	7.4	7.4			
rainbow® SpCO® Reusables			7.1	7.1		7.9	4.8	No Min	7.9	4.8	No Min
M-LNCS Disposables & Reusables						7.9	4.8	No Min	7.9	4.8	No Min

Patient Cable ▶		Red LNC						LNC			
LNCS® Sensor ▼	Parameter ▶	SpHb	SpMet	SpCO	ORi	RRp	PVi	SET®	RRp	PVi	SET®
LNCS Disposables & Reusables						7.9	4.8	No Min	7.9	4.8	No Min

[†] Masimo technology board software, sensor, and sensor cable combinations have been validated and tested for compliance per ISO 80601-2-61.

* Device must have SpHb parameter enabled.

** RRp is automatically provided on all versions of Rad-G

*** SpHb and SpCO measurements can be obtained simultaneously using the rainbow® Super DCI-mini sensor.

ORi, rainbow® Super DCI-mini sensor, RD rainbow Lite SET-1 sensors, and Rad-G have obtained CE Marking. Not available in the US.

rainbow® Super DCI-mini sensors and Rad-G are not licensed for sale in Canada. RD rainbow 4λ Sensors are not licensed for sale in Canada.

For professional use. See instructions for use for full prescribing information, including indications, contraindications, warnings, and precautions.

Masimo Americas
tel 1-877-452-7455
info-america@masimo.com

Masimo International
tel +41-32-720-1111
info-international@masimo.com

