



# Root<sup>®</sup> with Radius PCG<sup>™</sup>

Seamless Integration of Tetherless Mainstream Capnography

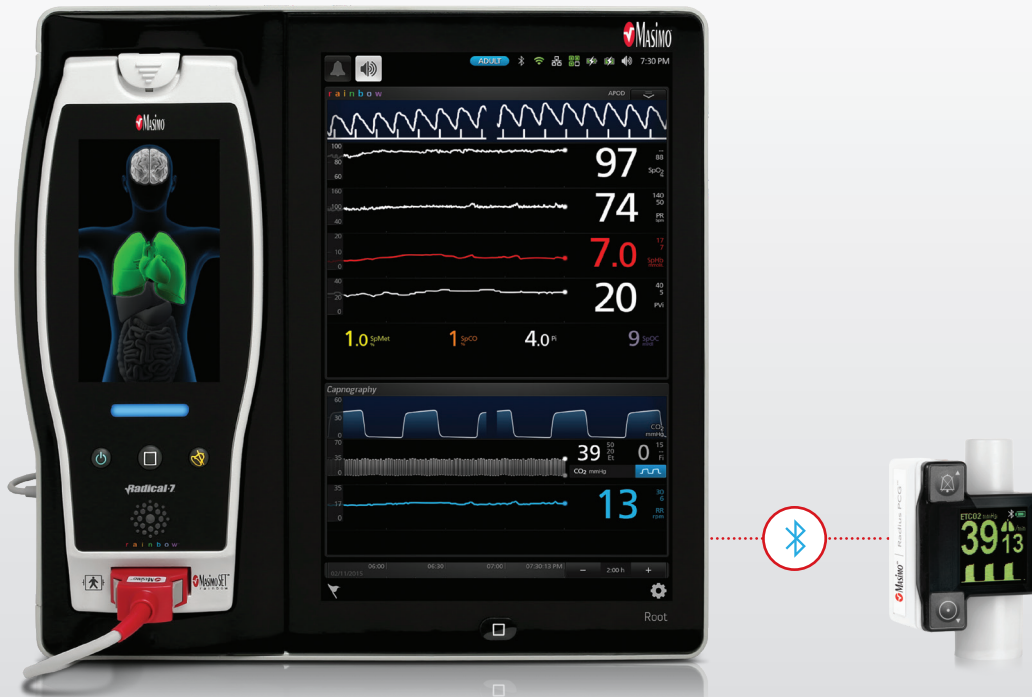

## Tetherless Solution

Cable-free design and quick Bluetooth<sup>®</sup> pairing facilitate easy movement between care areas, for all patient populations

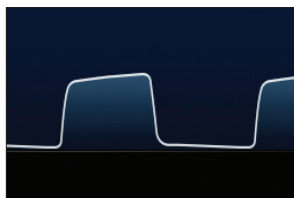

## Increased Visibility

High-resolution, secondary display of large, crisp waveforms and numbers improves visibility of waveform for better interpretation



## Real-time Trending

Clear trend display helps clinicians review progress over time and guide ventilation efforts



## Automated EMR Documentation

Electronic charting of patient data in the EMR via Masimo Patient SafetyNet<sup>™</sup>\* or Iris Gateway<sup>®</sup> simplifies workflow

## Supplemental Capnography Display on Root

Adjust trend display range to tailor monitoring needs for each patient's condition

Up to 96 hours of trend data enables bedside review of changes in a patient's status over time

Clearer, larger waveform display may assist clinicians in identifying wave patterns that suggest airway obstruction or tube dislodgement

Multi-touch screen allows quick adjustments to settings during care

Quick assessment in a dedicated, adjustable window



Audible and visual alarms for quick identification of alarming parameters



One-touch Bluetooth menu access for pairing or re-pairing to multiple Root monitors

## Pairing in Three Simple Steps

- 1 From the Root Bluetooth screen, select the Radius PCG Pair button
- 2 On Radius PCG, press the power button twice until the screen displays the Bluetooth symbol
- 3 Select Radius PCG on the Devices Found list on Root and touch the Pair button

## Ordering Information

ROOT	RADIUS PCG
Root .....9515	Radius PCG (mmHg) .....9961
Root with Noninvasive Blood Pressure and Temperature .....9695	Radius PCG (kPa) .....9962
	EMMA® Airway Adapter Adult/Paediatric .....17448
	EMMA Airway Adapter Infant .....17449

\* The use of the trademark Patient SafetyNet is under license from University Health System Consortium.

For professional use. See instructions for use for full prescribing information, including indications, contraindications, warnings, and precautions.

**Masimo U.S.**  
Tel: 1 877 4 Masimo  
info-america@masimo.com

**Masimo International**  
Tel: +41 32 720 1111  
info-international@masimo.com

